## ultrasonic nebulizer

### Anwendung

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### application

The ultrasonic-nebulizer micro converts electrical energy with a frequency of 1.7MHz into mechanical oscillation.

These high frequency oscillations cause the smallest aerosol particles to detach from the inhalation liquid.

The resultant aerosol particle size is below 5 microns which is small enough to deliver the aerosol into the respiratory tract, bronchi and alveoli.

The micro is designed for the delivery of all commonly inhaled medications in the treatment of various respiratory diseases and includes the delivery of hypertonic saline for bronchial provocation testing and for sputum induction.

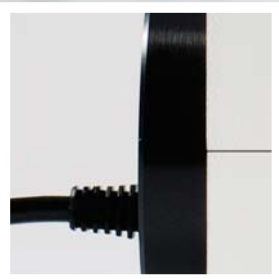
Smallest quantities (3-30 ml) of medicaments are nebulized except for a small residue.

Additional equipment like mouthpiece and mask support the specific treatment.

### hygiene

**micro** meets highest hygiene requirements due to the use of disposable articles like the dust filter and the bacteria filter for the carrying air.

All parts getting into contact with the patient and the aerosol are suitable for sterilization or disinfection respectively.

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## heated inhalation

Any ultrasonic-nebulizer type micro is available with connection for heating.

Two different kinds of heating are available:

## heating cable

The heating cable consists of an adapter with casted heating cable and connection cable with plug and is simply placed into the regular aerosol-tube.

## heating tube

The heating tube developed by us is simply used instead of the regular aerosol-tube and produces extraordinary carefully for the medicament a pleasant temperature of 37° C.

Due to the easy handling the heating tube is the variety which is more operator-friendly.

Both systems are fully sterilizable and meet highest hygiene standards.

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## the **micro** range

Our simple ordering system allows you to choose your equipment as required. All our systems are completely equipped available, ready for immediate application.

### **micro 800/800 U**

Incl. medicament-cup and bonnet for nebulizing smallest quantities from 3 ml on. Fitted with flexible support for aerosol tube, air-regulation-valve, autoclavable tubes, dust- and bacteria filter, mains cable

### **micro 801/801 U**

Incl. medicament-cup and bonnet for nebulizing smallest quantities from 3 ml on. Additional water-control-device complete with cannula for diverse bottle systems for continuous nebulizing. Fitted with bottle-support and support for aerosol tube, air-regulation-valve, autoclavable tubes, dust- and bacteria filter, mains cable

### **micro 802 / 802 U**

Incl. medicament-cup and bonnet for nebulizing smallest quantities from 3 ml on. Additional water-control-device complete with cannula for diverse bottle systems for continuous nebulizing. Fitted with mobile tripod, adjustable support, support for aerosol tube, bottle support, air-regulation-valve, autoclavable tubes, dust-

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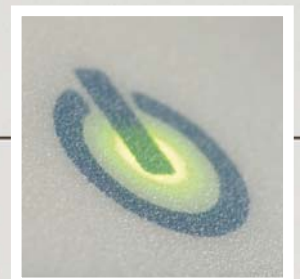
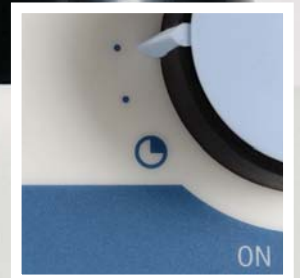
## handling

Due to easy handling and noiseless operation the **micro** range is also especially suitable for home care.

High operating security because of automatic frequency-swing and electronic prevention from operation without liquid makes this product easy to handle. Air stream and intensity are infinitely variable.

The self-regulating continuous feeding system makes the use of stock bottles of up to 2 litres possible.

The application of sterile-water systems is easily possible without change. All parts that need to be cleaned can be detached comfortably by means of a plug-in connection.

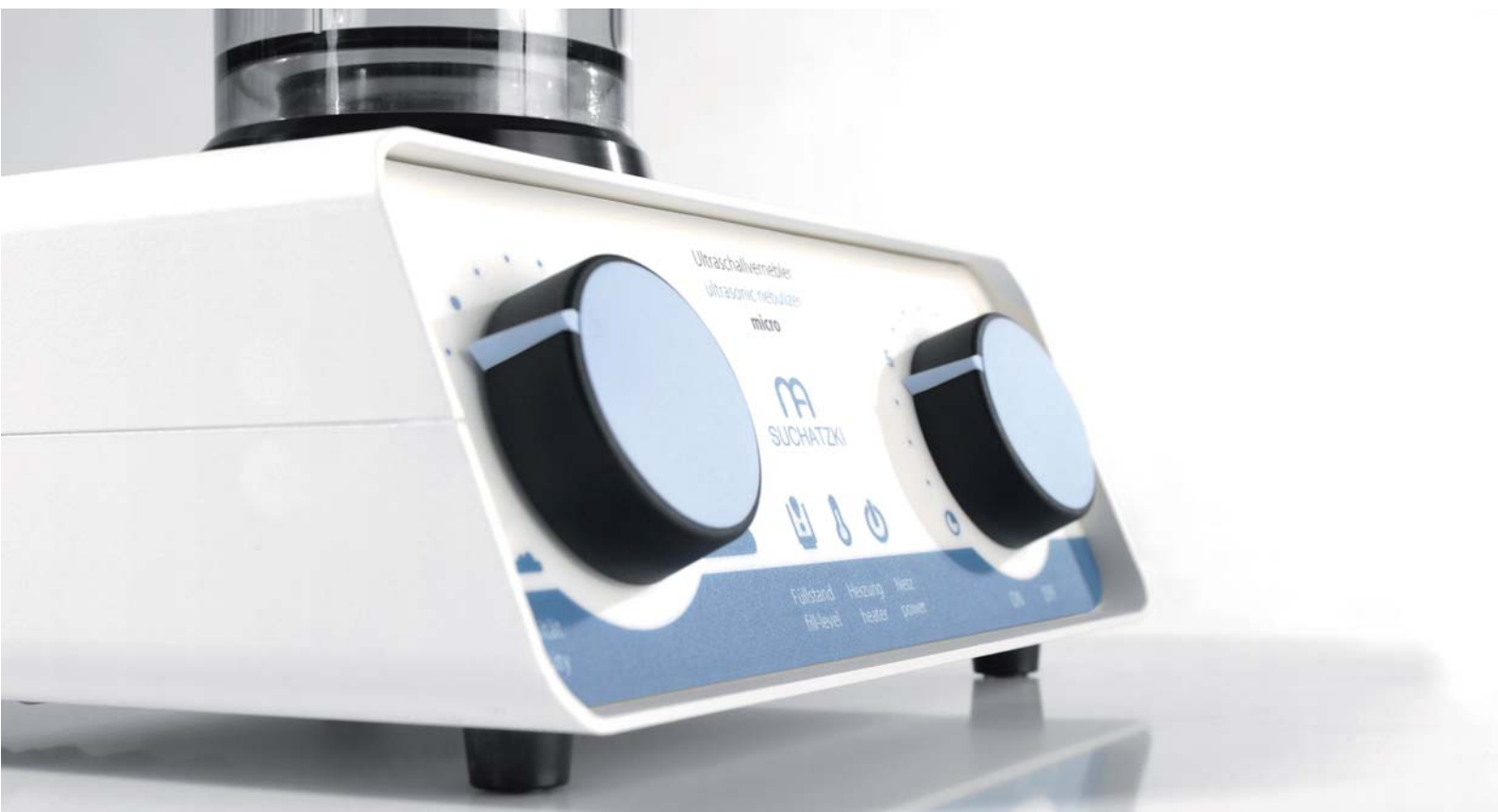
Med. Apparatebau Suchatzki  
Feldstrasse 2  
56477 Rennerod / Germany  
Tel. + 49 (0) 2664 8816  
Fax + 49 (0) 2664 6319  
info@suchatzki.com  
[www.suchatzki.com](http://www.suchatzki.com)

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